

Bio-Safe Bag Instructions


Important

Dear Doctor and Staff,

To keep the prescription work order forms free from contamination, please use dual-pocket Bio-Safe bags for shipping. One pocket can be sealed to isolate contaminated materials, such as impressions, from the prescription work order form.


Thank you for practicing the 7 steps of universal precautions for health safety. As a customer of Killian Dental, Bio-Safe bags are available free of charge. To request a supply, please call: (800) 317-7100.

1 Prepare Bag

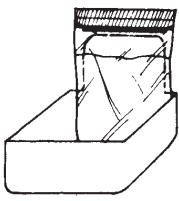


1. Place bag on case pan with flap over back side.

2 Open

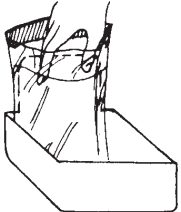


2. Spread case pocket open wide.

3 Insert Contaminated Materials

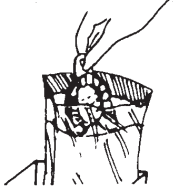


3. Insert contaminated items into bag's longer pocket.

4 Remove Liner Strip




4. Remove gloves, wash hands, peel off blue release liner to expose the adhesive strip.

5 Seal

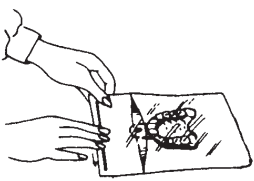


5. Pull taut. Press the adhesive strip to seal bag.

6 Keep other Materials Separate

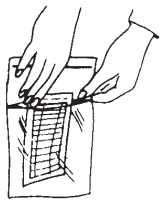

6. Insert Rx work order into outside pocket and tuck under flap.

7 Cushion any Stone Casts



7. Place bag in shipping box.

Clinical Aseptic Control Procedure for
isolation of biohazardous materials



DENTAL CERAMICS
67 Peters Canyon Rd., Irvine, Ca 92606
Telephone (800) 317-7100